


Drug of Abuse Reader Systems

Skannex, a world-wide leader in flexible and affordable Lateral Flow readers and software, introduces a new product line for reading Drug of Abuse tests. Based on Skannex ReaditLateral software, our new DOA product line provides your DOA-testing-customers with the most fully featured reader system on the market.

The Skannex DOA system is designed with advanced documentation of results, including printable test reports and a DataLog file which includes all test results. The test results can also be easily transferred to most data management programs.

The software is adaptable to read virtually any DOA cassette from single test to two sided multiple test products, and many flat panel cups. In addition, Skannex DOA software will reliably read adulterant pads included in many DOA products.


Main Screen

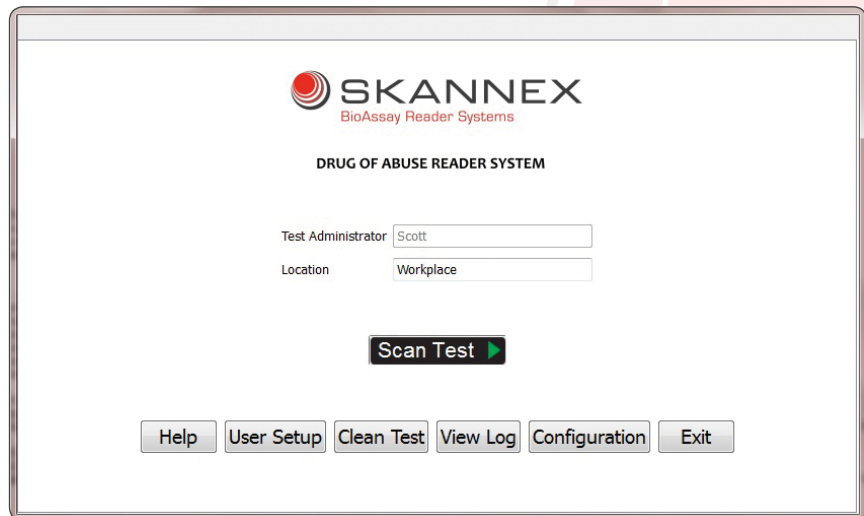

- Easy to use interface
- Graphics are easily adapted to reflect your company profile
- Single button read
- Software includes user login system with passwords

Results Screen

	Donor Name/ID	Test Kit	Test Kit Lot.	Temperature	Drug	Image	Software Result	Visual Result	Con
1	Normal	Read2	30505	3245-43	normal	COC	Negative	Negative	-
2						THC	Negative	Negative	-
3						AMP	Presumptive +	Presumptive +	-
4						OPI	Negative	Negative	-
5						PCP	Negative	Negative	-



28/09/2011

Help Print Report Save Result






- Results Screen provides full documentation of samples read, and allows user to visually interpret test results
- Test reports can be printed directly from Results Screen
- High quality image of test device is shown, and recorded with test results
- Data columns are easily customized for content

Various read options

Skannex software has the ability to read DOA cassettes in a number of ways, based on customer needs and regulatory constraints.






No read option

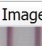- Software presents image of each strip and cassette read. Final results are determined only by entries made by the user

	Donor Name/ID	Test Kit	Test Kit Lot.	Temperature	Drug	Image	Visual Result	Comments
1	No Read	30505	3245-43	normal	COC		Negative	
2					THC		Negative	
3					AMP		Negative	
4					OPI		Negative	
5					PCP		Negative	

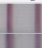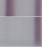



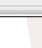
Software Flag option

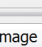- Final results are determined by user entries, but the software will “flag” positive results using pre-set negative / positive cut-off value

	Donor Name/ID	Test Kit	Test Kit Lot.	Temperature	Drug	Image	Visual Result	Comments
1	SW Flag 2	30505	3245-43	normal	COC		Negative	
2					THC		Negative	
3					AMP		Presumptive +	
4					OPI		Negative	
5					PCP		Negative	

Full Software Read option

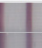- Software will determine individual strip results based on pre-set negative / positive cut-off value
- Software also allows the user to visually confirm test results

	Donor Name/ID	Test Kit	Test Kit Lot.	Temperature	Drug	Image	Software Result	Visual Result	Con
1	Normal Read2	30505	3245-43	normal	COC		Negative	Negative	
2					THC		Negative	Negative	
3					AMP		Presumptive +	Presumptive +	
4					OPI		Negative	Negative	
5					PCP		Negative	Negative	

Flexible product format

The Skannex DOA Software is installed on a standard PC connected to a quality Plustek scanner to become a fully functional reader system. Our new fully integrated reader, SkanFlexi, is another option for those who would like a ready to go professional instrument.

Additional information

Please visit our website for more information.

www.skannex.com



SkanFlexi